



PAD A CHEEK®

Mask Liner FFD

AirFit F10, AirFit F20, AirTouch F20

**Extra Padding Included,
tuck into the pocket if needed**



- Remove the headgear straps from the mask, leave the silicone cushion on.
- Place the arms of the mask through the slits (photo #2) with the Velcro on the hose connection side of the mask away from your face.
- Pull the pointed seam of the mask liner up over the top of the cushion.
- Pull the legs up and secure Velcro, blue magnet will show (Photos 3 and 4)
- Flip the mask over and smooth the liner over the cushion pulling all sides taut.
- Make sure the pad inside (to protect your nose bridge) is smooth. The mask should look like (Photo #6).
- Re-attach the headgear straps to your mask.

Make certain no exhalation vents are covered with the Mask Liner

Note: You may need to cut the hole larger to accommodate your nose or mouth, do this with sharp scissors a little bit at a time. It is very easy to get the hole too large.

These photos are with the AirFit F20 mask, but it fits both the F10 and the F20

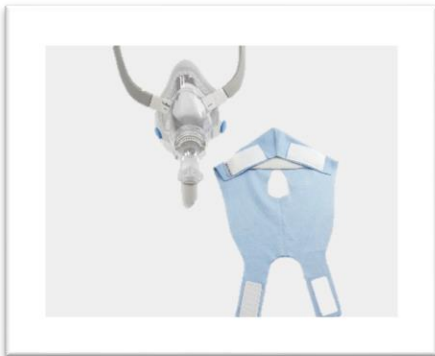


Photo 1



Photo 2



Photo 3



Photo 4

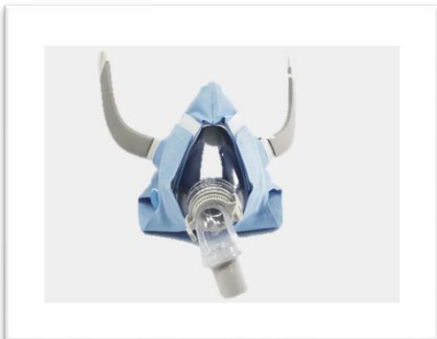


Photo 5

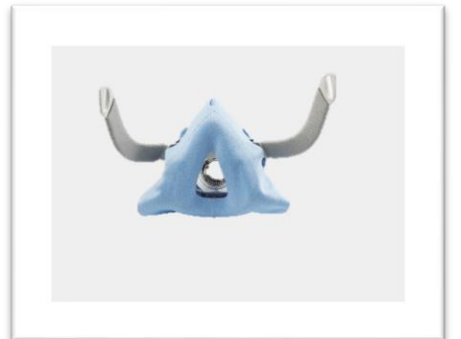


Photo 6

The PAD A CHEEK® Mask Liner is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed with warm water. If you have any questions, please contact me.

PAD A CHEEK® 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com

Designed and Sewn in the USA