



PAD A CHEEK[®]

Mask Liner



FFR

Ray Full Face Pediatric

- Remove the headgear from the mask and lay the **Mask Liner** out with the Velcro facing up (Photo #1).
- Lift up the sewn strap and slide the forehead cushion through (with the hole of the mask down toward the table) (Photo #2).
- Slide the sewn strap over the little knobs on the mask. (Photo #3)
- Pull the “legs” up and attach the Velcro loop sections to the hook sections (Photos #4).
- Put the forehead pad pockets over the mask forehead support and attach the straps by pushing the forehead part of the mask through the side holes in the Forehead Pad (Photo #5).
- Smooth the mask liner over the seal, tugging gently to remove wrinkles. (Photo #6). If the wrinkles are hard to remove, loosen the Velcro slightly.
- Reattach the straps to the mask

Make certain no exhalation vents are covered with the **PAD A CHEEK- Ray Mask Liner**.

Note: You may need to cut the hole larger to accommodate your nose, do this with sharp scissors a little bit at a time. **It is very easy to get the hole too large.**

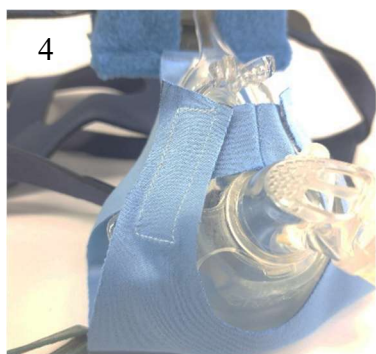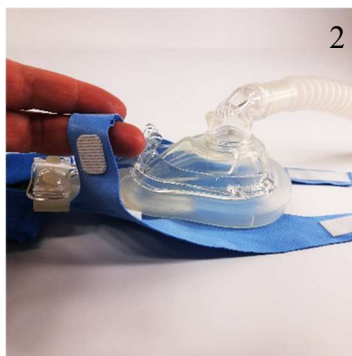

The **PAD A CHEEK Mask Liner** is made from 100% polyester (micro fleece and micro fiber), and can be machine or hand washed in warm water with facial soap. If you have any questions, please contact me.

PAD A CHEEK 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

website www.padacheek.com

Designed and Sewn in the USA