



PAD A CHEEK®

Mask Liner FFH



1. Lay the **PAD A CHEEK Mask Liner** and the mask itself out like the first photo.



2. Disconnect the magnets on both sides of the mask.

3. Place the lower part of the mask into the pocket of the mask liner so the magnets line up with the square holes as in the second photo.



4. Pull the upper part of the mask liner over the top of the mask cushion and attach the Velcro together as in the third photo.

5. Re-attach the magnets.



If the hole for your nose or mouth is not large enough for your comfort, trim the hole a little bit at a time to make it larger. If necessary, trim the rectangular hole near the vents.

Make certain no exhalation vents are covered with the Mask Liner

The **PAD A CHEEK® Mask Liner** is made from polyester micro fiber and should be hand washed with warm soapy water weekly and laid flat to air dry. If you have any questions, please contact us.

PAD A CHEEK® 434-985-4003

157 Sunset Drive, Stanardsville, VA 22973

email admin@padacheek.com

web site www.padacheek.com

Designed and Sewn in the USA